
# Actreen® Glys Set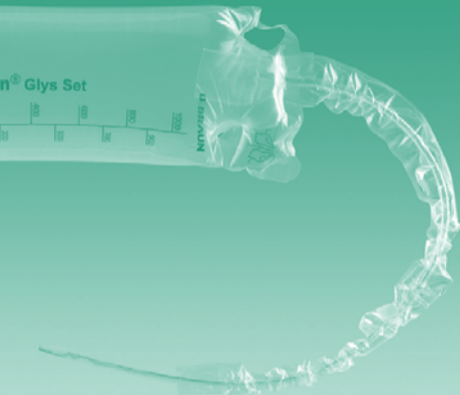

- EN** Instructions for use
- DE** Gebrauchsanweisung
- ES** Instrucciones de uso
- FR** Mode d'emploi
- IT** Istruzioni per l'uso
- NL** Gebruiksaanwijzing
- CS** Návod k použití
- DA** Brugsanvisning
- EL** Οδηγίες χρήσεως
- HU** Használati utasítás
- NO** Bruksanvisning
- PL** Instrukcja użycia
- PT** Instruções de uso
- RU** Инструкция по применению
- SK** Návod na použitie
- SV** Bruksanvisning

**B | BRAUN**

# Product range

		Charrière (Ø mm)	Ref
Actreen® Glyc Set Tiemann 45 cm, 30 Pcs/St.		CH08 (2,7 mm)	226108*
		CH10 (3,3 mm)	226110*
		CH12 (4,0 mm)	226112*
		CH14 (4,7 mm)	226114*
		CH16 (5,3 mm)	226116*

Actreen® Glyc Set Nelaton 50 cm, 30 Pcs/St.		CH08 (2,7 mm)	226208*
		CH10 (3,3 mm)	226210*
		CH12 (4,0 mm)	226212*
		CH14 (4,7 mm)	226214*
		CH16 (5,3 mm)	226216*
		CH18 (6,0 mm)	226218*

Actreen® Glyc Set Nelaton 25 cm, 30 Pcs/St.		CH06 (2,0 mm)	226306*
		CH08 (2,7 mm)	226308*
		CH10 (3,3 mm)	226310*
		CH12 (4,0 mm)	226312*
		CH14 (4,7 mm)	226314*
		CH16 (5,3 mm)	226316*

\* For each country a specific letter is added at the end of the reference

Actreen® Glyc Set is a ready-to-use sterile set containing a lubricated catheter connected to a graduated bag with an anti-reflux valve which can be emptied. Actreen® Glyc Set is indicated for intermittent urinary catheterisation.

- ④ In women, the urethra is short, about 4 cm long. Carefully locate the urethral opening, using a mirror at first.
- ④ In men, the urethra is longer and has curves that make catheterisation a little more difficult.

### Wash your hands very carefully. Make yourself comfortable.

- ④ Sit or stand, with the legs well apart in order to locate the urethral opening.
- ④ Sit or stand.

1 Wash the genitourinary area.

- ④ Take care to wash from the front to the back, beginning with the greater lips of the vagina and ending with the urethral opening.
- ④ First pull back the foreskin completely. Wash carefully both sides of the glans, and then the urethral opening.

2 Peel open the outer packaging and remove the catheter.

3 Remove the protective sleeves from around the catheter. You should now be left with just the sheath surrounding the catheter itself. Unfold the bag.

4 Unfold the top of the sheath and tear open on the perforated line.

5 The catheter is sterile and lubricated. Push the tip of the catheter out of the sheath a short distance. Do not touch the tip. You can hold the catheter through the sheath.

6 ④ Once the urethral opening has been located, introduce the catheter until it reaches the bladder.

④ Hold the penis upwards (1) to decrease curvature and facilitate catheter insertion. Once the catheter has been correctly positioned (2) the urine will drain.

7 The urine drains into the bag. Hold the catheter in place until the bladder has completely been drained. An eyelet is incorporated at the top of the bag, in order to hold it or hang it. Manual pressure on the lower abdomen towards the end of catheterisation will ensure that the bladder has been completely emptied.

8 After having gently withdrawn the catheter, the tear-off upper corner of the bag may be removed.

9 The bag may then be emptied.

10 At this stage, a urine sample may be collected for analysis. The set is then discarded according to the current local regulations.

⚠ Self-catheterisation should only be carried out under medical advice and on the order of a physician and only in accordance with the instructions given.

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B. Braun Medical A/S
B. Braun Medical AB