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III Disinfectant for haemodialysis monitors with powerful cleaning action

For professional use.

Active against bacteria, TbB, fungi, viruses (incl. HBV, HCV, HIV)\*. \*Tested acc. EN 1040/EN 13727, EN 1275/13624, EN 14348, RKI recommendation 01.2004 (viruses). Use: Disinfection of haemodialysis equipment: Follow manufacturers' instructions. Disinfection plans for tanks and machines with proportional pumps are available from the manufacturer. After each rinsing cycle, the dialysis unit should be tested for absence of disinfectant, using a 1% alcoholic phenolphthalein solution. The test solution must not show any red colouration. Information: Material compatibility: Not suited for materials sensitive to chlorine or alkaline. Caution! Do not mix with other products since dangerous gas (chlorine) could be released!

May be corrosive to metals. Causes severe skin burns and eye damage. Very toxic to aquatic life with long lasting effects. Keep only in original packaging. Avoid release to the environment. Wear protective gloves/protective clothing/eye protection/face protection/hearing protection. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water (or shower). IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. Dispose of contents/container to an approved waste disposal plant.

III Disinfectant for haemodialysis monitors with powerful cleaning action

For professional use.

Active against bacteria, TbB, fungi, viruses (incl. HBV, HCV, HIV)\*. \*Tested acc. EN 1040/EN 13727, EN 1275/13624, EN 14348, RKI recommendation 01.2004 (viruses). Use: Disinfection of haemodialysis equipment: Follow manufacturers' instructions. Disinfection plans for tanks and machines with proportional pumps are available from the manufacturer. After each rinsing cycle, the dialysis unit should be tested for absence of disinfectant, using a 1% alcoholic phenolphthalein solution. The test solution must not show any red colouration. Information: Material compatibility: Not suited for materials sensitive to chlorine or alkaline. Caution! Do not mix with other products since dangerous gas (chlorine) could be released!

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